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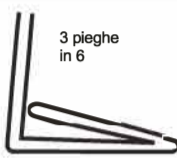
Revaginal

ovuli - ovules

Revaginal ovules. Medical device based on hyaluronic acid sodic salt, polycarbophil, lactic acid, Vitamin E, Vitamin A, 18-beta-glycyrrhetic acid, tea tree oil, phosphatidylcholine, semi-synthetic triglycerides.

Description:
Revaginal ovules is a medical device that helps maintain the natural defenses of the vagina and facilitates the restoration of vaginal flora; containing hyaluronic acid and lactic acid it helps delete the symptoms due to unbalanced vaginal hydration and/or vaginal pH. Hyaluronic Acid in its sodium salt form at a 10mg/ovule dosage exerts an hydrating and lubricating action against vaginal dryness contributes to restore a natural and comfortable vaginal environment.
Lactic Acid normally present in the vagina favors an immediate pH correction and facilitates the natural recovery of the delicate vulvo-vaginal ecosystem.
Polycarbophil interacting with vaginal mucus can constantly release H+ ions contributing to lower the pH of vaginal environment.

Indications:
Revaginal ovules are useful in the prevention and treatment of vaginal dryness to a specific character and in the case of atrophic vaginitis. Coadjuvant in case of irritation, burning and itching in the vaginal and vaginal disorders in infectious and inflammatory nature. Indicated in women of childbearing age, menopause as well as in perimenopause. The product is compatible with condoms.

Dosage:
Insert 1 ovule/day into the vagina - preferably in the evening before sleeping - for 5-7 days after the end of menstrual cycle or according to medical advice.

Instruction for Use:
1) Wash the vaginal area properly before application (preferably with both a vaginal wash external and internal);
2) Wash hands carefully before opening the blister extract one single ovule;
3) Gently insert the ovule deeply into the vagina, preferably in a supine position, avoiding accidental exit of the ovule out of the vagina.

Storage Conditions:
Store the package properly closed in a dry place, between 5°C and 25°C, away from light and heat sources.

Side effects:
Known hypersensitivity to any of the ingredients.

Warnings:
Keep out of reach and sight of children. Do not swallow. Do not use if package has been previously opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. Do not use the product in case of vaginal lesions and wounds. Do not use the product during menstruation. Do not exceed the suggested dosage without medical advice. The use - especially if prolonged - can lead to sensitiveness events. If this is the case, stop the treatment and consult the doctor in order to start a proper therapy. It is important to inform the doctor or the pharmacist of any undesired effects non reported in the insert leaflet.

Precautions on use:
During pregnancy, lactation and for children the use of ovules must be limited only if strictly required and under direct control of your doctor. In case of the intact hymen, before using consult your gynecologist to the specific conditions of use. Monouse device. In case of accidental exit of the ovule out of the vagina, do not re-use the ovule.

Presentation:
Folding box with two blisters containing 5 vaginal ovules each (10 ovules/pack) 2g/ovule.

Manufacturer:
Gruppo Farmaimpresa srl
Via Cipro, 1
25124 Brescia

The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.

FT.CE.15.11.F03 Rev. 23/11/2016