# Rigel

gel vaginale · vaginal gel

6 APPLICATORI MONODOSE DA 5 ML  
SINGLE DOSE APPLICATORS OF 5 ML

IDRATANTE E LENITIVO  
MOISTURIZING AND SOOTHING

# Rigel

gel vaginale · vaginal gel

#### Gel vaginale idratante e lenitivo.

##### INDICAZIONI

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#### Moisturizing and soothing vaginal gel.

##### INDICATIONS

Rigel vaginal gel is a medical device with pH 4.5 adjuvant in the prevention and treatment of external vaginal dryness. With its particular formulation it has a moisturizing, refreshing and soothing effect on the vulvo-vaginal area, persistent and prolonged in time. Rigel gel is indicated in cases of vaginal and vulvar dryness and in all situations where an effective soothing effect is needed which gives relief to the discomfort caused by poor intimate lubrication, such as irritation, itching, redness, burning and hypersensitivity phenomena. Hyaluronic acid has a moisturizing and emollient action against vaginal dryness, facilitating the restoration of a natural and pleasant vaginal environment. The lactic acid, a component normally present in the vagina, favors an immediate correction of the pH and facilitates the natural restoration of the delicate vulvo-vaginal ecosystem.

The formulation is enriched by the presence of vitamins with antioxidant action and by plant extracts which, due to their intrinsic properties, exert an emollient and soothing action.

##### HOW TO USE

We recommend applying the gel once a day.

##### COMPOSITION

Acqua, Glycerin, Triticum vulgare germ extract, Hydroxyethylcellulose, Lavandula angustifolia flower water, Imidazolidinyl urea, Polysorbate 20, Sodium hyaluronate, PEG-40 hydrogenated castor oil, Potassium sorbate, Xanthan gum, Lactic acid, EDTA, Glycine soja seed extract, Glycyrrhizic acid, Propylene glycol, Maltobextrin, Phenoxethanol, Ethylhexylglycerin, Retinyl palmitate, Tocopheryl acetate, Tetrasodium glutamate diacetate.

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**WARNING**  
Keep out of reach of children. Single-dose product to be used in a sterile administration. Store the product properly closed and away from light and heat sources. Do not use the product after the expiry date shown on the package. The expiry date is valid for the correctly stored product. Do not use the product if the package is damaged or opened. The device is not sterile. It is not a contraceptive. Do not use in case of organ injury. Do not swallow. Avoid contact with eyes. For pregnant and lactating women and girls, the use of the gel should be limited to cases of real need and under the direct supervision of the doctor. Do not use in case of individual hypersensitivity to the components present. Do not use the product in association with quaternary ammonium compound. The incorrect observation of the warnings, of the methods of use, as well as the use of the same single-dose applicator, could compromise the efficacy of the product or cause the aggravation of existing problems or cause the onset of problems that were not previously present. It is important to inform your doctor or pharmacist of any side effects not described in leaflet.

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MOISTURIZING AND SOOTHING GEL

#### INDICATIONS

Rigel vaginal gel is a medical device with pH 4.5 adjuvant in the prevention and treatment of external vaginal dryness. With its particular formulation it has a moisturizing, refreshing and soothing effect on the vulvo-vaginal area, persistent and prolonged in time. Rigel gel is indicated in cases of vaginal and vulvar dryness and in all situations where an effective soothing effect is needed which gives relief to the discomfort caused by poor intimate lubrication, such as irritation, itching, redness, burning and hypersensitivity phenomena. Hyaluronic acid has a moisturizing and emollient action against vaginal dryness, facilitating the restoration of a natural and pleasant vaginal environment. The lactic acid, a component normally present in the vagina, favors an immediate correction of the pH and facilitates the natural restoration of the delicate vulvo-vaginal ecosystem. The formulation is enriched by the presence of vitamins with antioxidant action and by plant extracts which, due to their intrinsic properties, exert an emollient and soothing action.

#### COMPOSITION

Aqua, Glycerin, Triticum vulgare germ extract, Hydroxyethylcellulose, Lavandula angustifolia flower water, Imidazolidinyl urea, Polysorbate 20, Sodium hyaluronate, PEG-40 hydrogenated castor oil, Potassium sorbate, Xanthan gum, Lactic acid, EDTA, Glycine soja seed extract, Glycyrrhethinic acid, Propylene glycol, Maltodextrin, Phenoxethanol, Ethylhexylglycerin, Retinyl palmitate, Tocopheryl acetate, Tetrasodium glutamate diacetate.

#### HOW TO USE

We recommend applying the gel once a day. Before applying, wash your hands and wash the affected area after which:

- Remove the cap at the end of the cannula of the single-dose applicator.
- Press the single-dose applicator until the tube is completely empty.
- Apply the gel and massaging gently on the affected area.
- Discard the single-dose applicator.

Do not store the product residue for later use.

#### WARNINGS

Keep out of reach of children. Single-dose product to be used in a single administration. Store the product properly closed and away from light and heat sources. Do not use the product after the expiry date shown on the package. The expiry date is valid for the correctly stored product. Do not use the product if the package is damaged or opened. The device is not sterile. It is not a contraceptive. Do not use in case of organ injury. Do not swallow. Avoid contact with eyes. For pregnant and lactating women and girls, the use of the gel should be limited to cases of real need and under the direct supervision of the doctor. Do not use in case of individual hypersensitivity to the components present. Do not use the product in association with quaternary ammonium compounds. The incorrect observance of the warnings, of the methods of use, as well as the re-use of the same single-dose applicator, could compromise the efficacy of the product or cause the aggravation of existing problems or cause the onset of problems that were not previously present. It is important to inform your doctor or pharmacist of any side effects not described in this leaflet. External use.

#### SIDE EFFECTS

In normal and reasonably foreseeable conditions of use, undesirable effects specifically related to the use of the product are not known at the moment. Its compatibility has been verified, but in case of irritating or allergic phenomena, suspend its use and contact your doctor / gynecologist.

#### PACKAGING

6 single dose applicators of 5 ml each.

Manufacturer:  
Gruppo Farmaimpresa Srl  
Via Cipra, 1 - 25124 Brescia - ITALY

The Medical Device must not be disposed in the wastewater or household waste. Ask your pharmacist how to dispose medical devices no longer required. This will help protect the environment.

FT.CE.1271.F00  
Rev00 11/2020