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IDANIL is a medical device indicated in the processes of anorectal disorders such as external hemorrhoids and situations of anal itching or burning. Its formulation is able to form an invisible film with a barrier effect, capable of restoring the correct level of hydration and the natural welfare of the mucous membranes. HEMO GUARD protects against irritation and perianal congestion, allowing quick relief from the sensation of discomfort.

Dosage: Apply a small amount of product on the affected area 3-4 times a day.

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**MEDICAL DEVICE INDICATED IN THE
PROCESSES OF ANORECTAL DISORDERS**

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Warning: Keep out of the reach of children. For How to use, Warnings, Interactions and incompatibilities and Side effects read carefully the leaflet.

**Fabbricante /
Manufacturer:**
Gruppo Farmaimpresa srl - Via Cipro, 1 - 25124 Brescia - Italy

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FT.CE.129.F02 Rev.31/05/2024

IDANIL EMULGEL

MEDICAL DEVICE INDICATED IN THE PROCESSES OF ANORECTAL DISORDERS

30 ml

IDANIL is a medical device indicated in the processes of anorectal disorders such as external hemorrhoids and situations of anal itching or burning. Its formulation is able to form an invisible film with a barrier effect, capable of restoring the correct level of hydration and the natural welfare of the mucous membranes. IDANIL protects against irritation and perianal congestion, allowing quick relief from the sensation of discomfort.

Composition: Aqua, Malva sylvestris leaf water, Aloe vera gel, Glycerin, Escin, Melilotus ocnalis extract, Phenoxyethanol, Ethylhexylglycerin, Tetrasodium glutamate diacetate, Potassium sorbate, Citric acid, Propylene glycol, Chamomilla recutita flower water, Glucose monohydrate, Sodium Lactate, Sodium PCA, Glycine, Fructose, Urea, Niacinamide, Inositol, Sodium benzoate, Lactic acid, Hamamelis virginiana, Sorbitol, Benzyl alcohol, Hydrolyzed wheat protein, Panthenol, Menthyl lactate, PEG-40 hydrogenated castor oil, Acrylates/c10-30 alkyl acrylate crosspolymer, Carbomer, Prunus amygdalus dulcis oil, Tocopherol mix, Tocopheryl acetate, Bisabolol, Propylene glycol dicaprylate/dicaprate, Oryza sativa (rice) bran oil, Tocotrienols, Triethanolamine, Menthol, PPG-26-Buteth-26, Trideceth-9, Ethoxydiglycol.

Dosage and How to use: Apply a small amount of product on the affected area 3-4 times a day.

- 1) Carefully wash the affected areas before application and dry gently.
- 2) Open the tube and screw the cannula contained in the package.
- 3) Squeeze the tube until the cannula fills with gel.
- 4) Squeeze the tube until a sufficient amount of product is applied.
- 5) Apply to the external area.
- 6) Remove the cannula, wash it thoroughly and store it.
- 7) Make sure that the tube is closed properly.

After application it is advisable to avoid evacuation for at least 20-30 minutes to avoid early removal of the product. Renew the treatment periodically as needed and always after washing operations.

Storage conditions: Store the product at a temperature between 5°C and 25°C away from lights and heat sources. Keep the product in the original packaging.

Warnings: The device is not subject to special precautions for use, however it is recommended to:

- External Use
- Do not use in case of wounds or injuries of the anal and perianal area.
- Carefully wash the cannula after each application.
- Keep out of the reach of children;
- Do not exceed the recommended dose;
- Do not use after the expiry date; the expiry date refers to the intact and properly stored product;
- In case of adverse reactions, suspend the use of the product and consult your doctor in order to fix a proper therapy;
- Do not use the product if the package is not intact;
- If used during pregnancy, lactation and in children or concurrent use with drugs consult your doctor.
- In case of use during pregnancy, breastfeeding and on children, consult your doctor.
- The incorrect observance of the warnings, of the methods of use, as well as the use of the product by more than one person, could compromise the efficacy of the product or cause the aggravation of existing problems or cause the onset of problems that were not previously present.

It is important to report to the doctor or pharmacist the appearance of undesirable effects not mentioned in this leaflet.

Interactions and incompatibilities: No interactions with other drugs or incompatibilities with other products used in the treatment of hemorrhoids or anal fissures are known or reported.

Side effects: Under normal and reasonably foreseeable conditions of use, no undesirable products specifically related to the use of the product are currently known. Do not use in case of known hypersensitivity to the product components.

Presentation: Gel in 30 ml tube with multiuse cannula.

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The medical device must not be thrown into waste water or household waste. Ask the pharmacist how to dispose of the medical devices that are no longer used. This will help protect the environment.