XY  
AQUAGEL

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Its formulation has been studied to:

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**How to use:**

The product should be used during the ovulation phase, especially during the most fertile days of the period (13<sup>th</sup> - 14<sup>th</sup> - 15<sup>th</sup> day). Introduce 5 minutes before sexual intercourse.

- Break the closure seal at the end of the cannula of the single-dose applicator
- Insert the single-dose applicator in the vaginal canal
- Press the single-dose applicator until the tube is completely empty
- Discard the single-dose applicator

It is advisable to apply the product once a day or more often according to the doctor's instructions.

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Acqua, Glucosio, Fruttosio, Sodio Ialuronato, Magnesio Citrato, Calcio Piruvato, Sorbitolo, Acido Lattico, Acido Citrico, Inositol, Potassio Citrato, Sodio Bicarbonato, Zinco Gluconato, Carnitina, Creatina, Sodio Cloruro, DMDM Idantoina, Sodio Idrato, Fenossietanolo, Etilxilglicerina, Idrossietilcellulosa.

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**⚠ Warnings:**

- Do not use in case of damaged mucous membranes, post-surgery or acute vaginal infection.
- Do not swallow.
- Do not use in case of individual hypersensitivity ascertained to the components present into the product.
- Do not use the product in conjunction with quaternary ammonia compounds.
- It's not a contraceptive.
- XY AQUAGEL has no spermicidal activity.
- Carefully read the insert leaflet.

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FT.CE.109.F.00 - Rev.00 del 19/12/2019

**XY**  
AQUAGEL

**XY AQUAGEL is a vaginal lubricant that can be used in cases where conception is desired.**

*Its formulation has been studied to:*

- Create an optimal environment for spermatozoa
- Contribute to maintaining a pH value suitable for natural conception
- Obtain an osmolarity compatible with the seminal fluid.

*XY Aquagel is a gel that does not stain, is formulated with preservatives compatible for intended use, is parabens, silicones and mineral oils free.*

**How to use:**

The product should be used during the ovulation phase, especially during the most fertile days of the period (13<sup>th</sup> -14<sup>th</sup> -15<sup>th</sup> day). Introduce 5 minutes before sexual intercourse.

- Break the closure seal at the end of the cannula of the single-dose applicator
  - Insert the single-dose applicator in the vaginal canal
  - Press the single-dose applicator until the tube is completely empty
  - Discard the single-dose applicator
- It is advisable to apply the product once a day or more often according to the doctor's instructions.

**Composition:**

Aqua, Glucose, Fructose, Sodium Hyaluronate, Magnesium Citrate, Calcium Piruvate, Sorbitol, Lactic Acid, Citric Acid, Inositol, Potassium Citrate, Sodium Bicarbonate, Zinc Gluconate, Carnitine, Creatine, Sodium Chloride, DMDM Idantoin, Sodium hydroxide, Phenoxyethanol, Ethilexylglycerin, Hydroxyethylcellulose.

**Warnings:**

- Keep out of reach of children.
- Single-dose applicators: the content of each tube should be used in a single administration
- Store the product properly sealed and protected from light and heat sources.
- Do not use the product after the expiry date reported on the packaging.
- The expiry date is valid for the product properly stored.
- Do not use the product if the package is damaged or opened.

- The device is not sterile.
- It's not a contraceptive.
- XY AQUAGEL has no spermicidal activity.
- Do not use in case of damaged mucous membranes, post-surgery or acute vaginal infection.
- Do not swallow.
- Avoid contact with eyes.
- Do not use in case of individual ascertained hypersensitivity to the components present into the product.
- Do not use the product in conjunction with quaternary ammonia compounds.
- The incorrect observance of the warnings, of the methods of use, as well as the re-use of the same single-dose applicator, could compromise the efficacy of the product or cause the aggravation of existing problems or cause the onset of problems that were not previously present.
- For lactating women, the use of the gel should be limited to cases of real need and under the direct supervision of the doctor.
- Do not use the product for a period exceeding 30 consecutive days.
- It is important to inform your doctor or pharmacist of any side effects not described in this leaflet.

**Side effects:**

Under normal and reasonably foreseeable conditions of use any undesirable effects specifically related to the use of the product are currently unknown. Its compatibility has been verified, but in case of irritating or allergic phenomena, stop to use it and contact your doctor/gynecologist.

**Packaging:**

6 single dose applicators of 5 mL each one.

 **Manufacturer:** Gruppo Farmaimpresa srl  
Via Cipro, 1 – 25124 Brescia - Italy

 The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.