Con Acido Borico
e Acido Lattico
With Boric Acid
and Lactic Acid

Schiuma / Foam
Revaginal

Revaginal
Schiuma
Foam

Con Acido Borico
e Acido Lattico
With Boric Acid
and Lactic Acid

PJ
PHARMA 150ml

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Con Acido Borico
e Acido Lattico
With Boric Acid
and Lactic Acid

PJ
PHARMA 150ml

Revaginal Foam is a ready-to-use cleaning solution. Useful for routine hygiene and intimate protection, as well as an excellent substitute for intimate cleansers when it is not possible to resort to the usual hygiene practices. Formulated for vulvo-perineal daily hygiene, it helps restore the physiological pH of the intimate areas. It is a medical device particularly indicated as an adjunct in the treatment and prevention of small intimate discomforts such as itching, redness, light burns, hypersensitivity phenomena and in case of vaginal ailments even of a mycotic nature. **Revaginal Foam** is based on lactic acid, a component normally present in the vagina, and boric acid. These components favor an immediate pH correction, facilitating the natural restoration of the delicate vulvo-perianal system and helping to make it unfavorable to bacterial and / or fungal proliferation.

How to use: it is recommended to use the foam once a day. Before application, wash your hands and shake the bottle. Using the dispenser, pour a dose into the palm of your hand by gently distributing the foam from the external genitals to the perianal area. Leave to act for a few moments then remove residues with a clean towel or rinse.

Warnings: Keep out of reach of children. Read the package leaflet carefully before use.

Attention: for warnings and side effects, read the package leaflet carefully

Composition: Tegobetaine, Boric Acid, Sodium Lactate, Sodium PCA, Glycine, Fructose, Urea, Niacinamide, Inositol, Sodium Benzoate, Lactic Acid, Polysorbate, Water, Perfume, PEG-40 Hydrogenated Castor Oil, Disodium EDTA, Potassium Sorbate, wheat protein, Chlorhexidine Digluconate, Glycolic Acid.

Fabbricante / Manufacturer:
Gruppo Farmaimpresa srl
Via Cipro, 1- 25124 Brescia (Italy)

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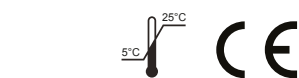
Revaginal Foam

With Boric Acid and Lactic Acid.

Revaginal Foam is a ready-to-use cleaning solution. Useful for routine hygiene and intimate protection, as well as an excellent substitute for intimate cleansers when it is not possible to resort to the usual hygiene practices. Formulated for vulvo-perineal daily hygiene, it helps restore the physiological pH of the intimate areas. It is a medical device particularly indicated as an adjunct in the treatment and prevention of small intimate discomforts such as itching, redness, light burns, hypersensitivity phenomena and in case of vaginal ailments even of a mycotic nature.

Revaginal Foam is based on lactic acid, a component normally present in the vagina, and boric acid. These components favor an immediate pH correction, facilitating the natural restoration of the delicate vulvo-perianal system and helping to make it unfavorable to bacterial and / or fungal proliferation.

How to use: it is recommended to use the foam once a day. Before application, wash your hands and shake the bottle. Using the dispenser, pour a dose into the palm of your hand by gently distributing the foam from the external genitals to the perianal area. Leave to act for a few moments then remove residues with a clean towel or rinse.

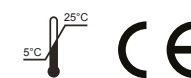
Composition: Tegobetaine, Boric Acid, Sodium Lactate, Sodium PCA, Glycine, Fructose, Urea, Niacinamide, Inositol, Sodium Benzoate, Lactic Acid, Polysorbate, Water, Perfume, PEG-40 Hydrogenated Castor Oil, Disodium EDTA, Potassium Sorbate, wheat protein, Chlorhexidine Digluconate, Glycolic Acid.

Warnings: Keep out of reach of children. Keep the product properly sealed and protected from light and heat sources. Do not use the product after the expiry date shown on the package. The expiry date is valid for the product properly stored. Do not use the product if the package is damaged or opened. The device is not sterile. Do not use in case of organ injuries. Do not swallow. Avoid contact with eyes. For pregnant and lactating women, the use of the product must be limited to cases of actual need and under the direct supervision of the doctor. Do not use in case of individual hypersensitivity to the components present. The incorrect observance of the warnings, the instructions for use, as well as the use of the same bottle by several people, could compromise the effectiveness of the product or cause existing problems to worsen or cause problems to arise before not present. Do not use the product on children under 3 years of age. It is important to report the appearance of undesirable effects not described in this package leaflet to the doctor or pharmacist.

Side effects: In normal and reasonably foreseeable conditions of use, no undesirable effects specifically related to the use of the product are currently known. Its compatibility has been verified, but in the event of irritative or allergic phenomena, stop the use and contact your doctor / gynecologist.

Packaging: 150 mL bottle

 **Manufacturer:**
Gruppo Farmaimpresa srl
Via Cipro, 1- 25124 Brescia (IT)

 The Medical Device must not be disposed in the waste water or household waste. Ask your pharmacist how to dispose Medical Devices no longer required. This will help protect the environment.