Revaginal

Lavanda

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Revaginal

Vaginal Douche

REVAGINAL VAGINAL DOUCHE, protective formulation of the vaginal mucosa, is a medical device that contains hyaluronic acid, polysaccharide already present in the human body tissues which, thanks to its film-forming and protective features, favors a natural process of cell regeneration. Hyaluronic acid content in Revaginal douche has a low molecular weight and a low viscosity; this hyaluronic acid allows a high protective activity.

Indications: The formulation is indicated as a preventive purpose to preserve the integrity of the vaginal and vulvar mucosa. Useful for the treatment of symptoms of vulvovaginitis characterized by small vaginal discharge, itch, irritation, burning and vulvar pain. Revaginal vaginal douche gives a pleasant sensation of freshness and cleanliness.

Composition: Sodium Hyaluronate, Lactic Acid, PVP, Disodium EDTA, Potassium Sorbate, Sodium Benzoate.

How to use: An irrigation per day or every two days, depending on the severity of the illness, even in the presence of irritation of the vaginal mucosa. Wash hands properly before applying. Shake before use. Break the capsule bottle. Remove the cannula from its packaging. Apply the single-use cannula on the bottle. Insert the cannula into the vagina and gently press the bottle. Let discharge gradually all the solution inside the vagina. Remove the cannula and throw it into the trash bin. Dry off after finishing.

Side effects: Hypersensitivity to some components. The use, especially if prolonged, can cause episodes of sensitization; if this occurs, suspend treatment and consult doctor to establish an adequate therapy. It is important to inform your doctor or pharmacist if you notice side effects that are not listed in this leaflet.

Warnings and precautions for use: Keep out of reach of children. Single-use: reuse or use by more people of the same bottle, content or cannula, may involve the non efficacy of the product, the intensification of existing problems and the occurrence of problems previously not present. Use the entire content of the bottle for each application. Do not swallow. Read carefully the instructions for use. Do not use the product in case of vaginal injuries or wounds. Do not use in the presence of menstruation. Do not use the product if the packaging does not show intact. Do not use the product after the expiration date; expiration date refers to the product intact and well preserved. For pregnant women and for the little girls the use of douche must be limited to cases of real need and under direct medical supervision. In case of intact hymen, before using consult your gynecologist about the specific methods of use.

Storage conditions: Keep packaging well closed at room temperature (5°C - 25°C) away and far from light and heat sources.

Presentation: It contains 4 bottles of 30ml and 4 dispensing single use cannulae.

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The Medical Device must not be disposed of via wastewater or household waste. Ask your pharmacist how to dispose of medical devices no longer required. This will help protect the environment.

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REVAGINAL VAGINAL DOUCHE, perfumed formulation of the vaginal mucosa, is a medical device that contains hyaluronic acid, polyacrylamide glycolic and other active ingredients, as well as a controlled delivery system. The hyaluronic acid content in REVAGINAL DOUCHE has a low molecular weight and a low viscosity. This hyaluronic acid allows a high permeability activity.

Based on: Sodium Hyaluronate, Lactic Acid, PVP, Oxidized EDTA, Potassium Sorbate, Sodium Benzoate.

INDICATIONS: The formulation is indicated as a preventive purpose to promote the integrity of the vaginal and vulvar mucosa. Useful for the treatment of symptoms of vulvovaginitis characterized by anal vaginal itching, perineal, vaginal, irritation, burning and odor. Revaginal vaginal douche gives a pleasant sensation of freshness and cleanliness.

HOW TO USE: An injection per day or every two days, depending on the severity of the inflammation, even in the presence of irritation of the vaginal mucosa. Wash hands properly before applying. Shake before use. Break the capsule bottle. Remove the cannula from its packaging. Apply the hyaluronic cannula on the body, insert the cannula into the vagina and gently press the bottle. Let discharge gradually all the solution into the vagina. Remove the cannula and throw it into the trash bin. Dry off after finishing.

WARNINGS: Keep out of reach of children. Do not swallow. Carefully read the instructions for use.

STORAGE CONDITIONS: Keep product well sealed at room temperature (5°C - 25°C) and far from heat sources.

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