Rinofast
Spray nasale
Nasal spray
20 ml

FT/CE/12/A08L21P
Rev. del 24/05/2016

RINOFAST FLU - 20ml
Spray Nasale / Nasal Spray

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Rinofast *flu*

Nasal spray 20ml

Isotonic saline solution at controlled pH with N-Acetyl Cysteine and Methyl-Sulfonil Methane (MSM) indicated for daily nasal hygiene. Against the irritation caused by external agents resulting from environmental pollution. Helps to keep free the nose, reduces the dryness moisturizing surface gently. Helps regulate nasal secretions after rhinorrhea and prevent the formation and construction of bacterial biofilm.

INGREDIENTS:

Aqua, Sodium chloride, N-Acetyl Cysteine, Methyl-Sulfonil Methane (MSM), Sodium Bicarbonate, Sodium Ascorbyl Phosphate, Ethylenediaminetetraacetic acid Disodium salt, Potassium sorbate, Sodium benzoate, Aroma.

DOSAGE:

2-3 spray in each nostril, 3-4 times a day, or more often as needed.

INSTRUCTIONS FOR USE:

1. Remove the protective cap from the dispenser.
2. Hold the bottle and the head vertically, gently place the nozzle of the dispenser at the entrance of the nostril.
3. Press with two fingers on the basis of the dispenser.
4. Repeat with the other nostril.
5. Clean the dispenser.
6. Replace the protective cap dispenser.

WARNINGS:

Keep away from the reach of the children. In some cases the nebulization can cause a mild and transient sense of local irritation. Avoid spraying in eyes. The use of the same bottle, by multiple subjects, can facilitate the transmission of infections. Do not use in case of epistaxis in progress or just completed. Prohibited use by children under 3 years of age.

STORAGE:

Store the product at room temperature away from light and heat sources. Do not use the product after the expiration date or if the packaging appear sun damaged. The expiry date refers to the product properly packaged and correctly stored.

SIDE EFFECTS:

Possible hypersensitivity to any of the components. The use of the product, especially if prolonged, may induce sensitivity phenomena. In this case stop the treatment and consult your doctor for a proper therapy. It is important to inform your doctor or pharmacist of any other undesired effects not reported in this insert leaflet.

PRESENTATION:

Solution in 20 ml glass bottle with spray dispenser.

Manufacturer:
Gruppo Farmalimpresa srl
Via Cipro, 1
25124 Brescia Italy

The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.

FT/CE/12/F04 Last Rev. del 18/11/2015