20 ml
PHARMA
AG FLU
SPRAY GEL

AG FLU SPRAY GEL

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SPRAY GEL

20 ml

MEDICAL DEVICE USEFUL TO FAVOR THE HYDRATION OF NASAL MUCOSA

AG FLU SPRAY GEL is a medical device useful to promote the hydration of the nasal mucosa, the fluidification of nasal secretions and consequently their removal. It is indicated in all nasopharyngeal diseases characterized by congestion and alteration of mucociliary clearance. It helps to keep the nose free, inhibiting the adhesion of pathogenic bacteria, thereby preventing the development of infections in the upper respiratory tract.

Composition: Water, Sodium chloride, N-acetyl-cysteine, Xylitol, Colloidal silver, Sorbitol, Sodium benzoate, Potassium sorbate, Citric acid, Sodium phosphate, Sodium hydroxide, Acacia gum, Xanthan gum, Mint flavor.

Posology: 2-3 spray per nostril, 3-4 times a day, or more often as needed.

How to use:

- 1) Remove the protective cap from the dispenser.
- 2) Hold the bottle and the head vertically, gently position the spout at the mouth of the nostril.
- 3) Press with two fingers on the base of the dispenser.
- 4) Repeat the operation with the other nostril.
- 5) Clean the dispenser.
- 6) Put the dispenser protection cap back on.

Storage conditions: Store the product at a temperature between 5°C and 25°C, away from light and heat source. Keep the product in the original packaging. Do not use the product after the expiry date or if the package is not intact. The expiry date refers to intact and correctly stored product. The product can be stored for a maximum of 30 days after first opening.

Warnings: Keep out of the reach and sight of children. Do not use the product in children under three years of age. In some cases, nebulization can cause a slight and transient sense of local irritation. Avoid spraying in the eyes. The use of the same bottle, by several subjects, can facilitate the transmission of infections. Not intended for oral use. Do not use in case of nosebleeds in progress or just ended. Do not use if the nasal mucosa is not intact. In the event of adverse reactions, discontinue use of the product and contact your doctor. For the use of the product during pregnancy, breastfeeding or in children, it is advisable to consult a doctor. The incorrect observance of the warnings and methods of use could affect the effectiveness of the product or cause the aggravation of existing problems as well as cause the occurrence of problems not previously present. The product contains Colloidal Silver, do not use it in case of ascertained sensitivity towards the component.

Side effects: Under normal and reasonably foreseeable conditions of use, no undesirable effects specifically related to the use of the product are currently known. Do not use in case of hypersensitivity to the components. Prolonged use can give rise to sensitization phenomena: where this occurs, stop treatment, and consult your doctor in order to establish appropriate therapy. If you experience any undesirable effects not described in this package leaflet, please inform your doctor or pharmacist.

Presentation: Solution in glass bottle of 20 ml with vertical spray erogator.

 **Manufacturer:**
Gruppo Farmaimpresa srl
Via Cipro, 1 - 25124 Brescia - Italy

The medical device must not be thrown into wastewater or household waste. Ask your pharmacist how to throw away the Medical Devices you no longer use. This will help protect the environment.

FTCE180.SP/F00 Rev. 0 del 20/05/2021