
Product: **REVAGINAL GEL**
 Internal code:
 Country:
 Company:
 Operator:

Product: REVAGINAL GEL
Internal code:
Country:
Company:
Operator:

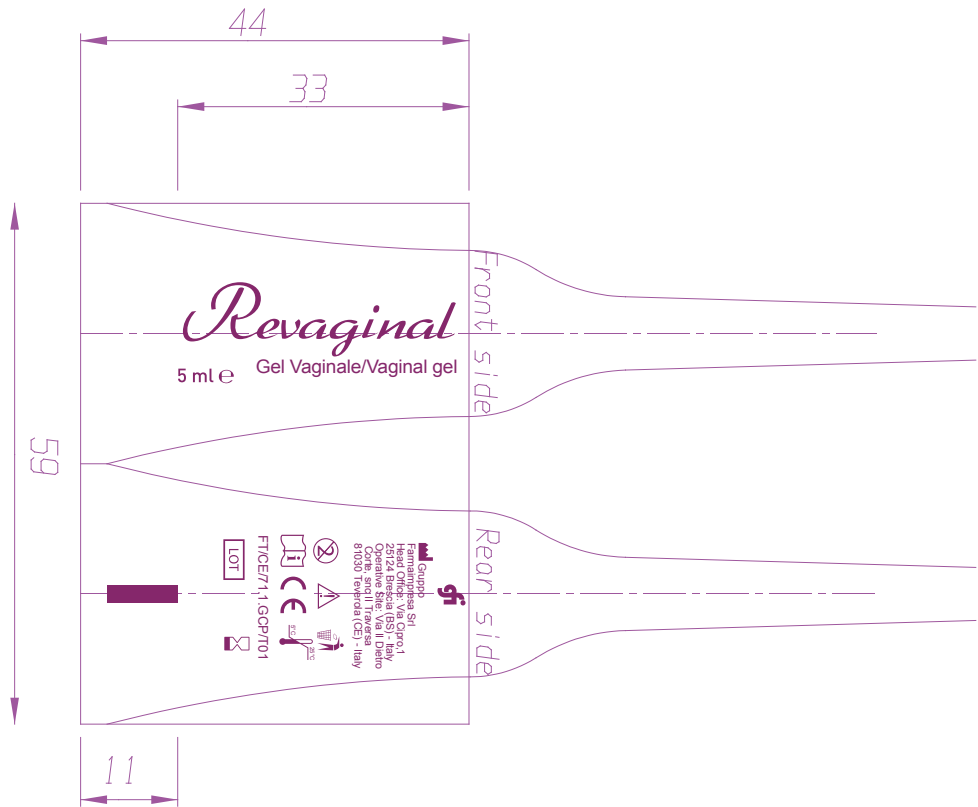Product: **REVAGINAL GEL**
 Internal code:
 Country:
 Company:
 Operator:

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Moisturizing and soothing vaginal gel.

Indications:

Revaginal vaginal gel is a medical device with pH 4.5 adjuvant in the prevention and treatment of vaginal and vulvar dryness. With its particular formulation it has a moisturizing, refreshing and soothing effect on the vulvo-vaginal area. Revaginal gel is indicated in cases of vaginal and vulvar dryness and in all situations where an effective soothing effect is needed which gives relief to the discomfort caused by poor intimate lubrication, such as irritation, itching, redness, burning and hypersensitivity such as in case of menopause or if you are taking estrogen-progestins. Hyaluronic acid has a moisturizing and emollient action against vaginal dryness, facilitating the restoration of a natural and pleasant vaginal environment. The lactic acid, a component normally present in the vagina, favors an immediate correction of the pH and facilitates the natural restoration of the delicate vulvo-vaginal ecosystem. The formulation is enriched by the presence of vitamins with antioxidant action and by plant extracts which, due to their intrinsic properties, exert an emollient and soothing action.

Composition:

Aqua, Imidazolidinyl Urea, Disodium EDTA, Glycerin, Aloe Barbadensis Leaf Juice, Potassium Sorbate, Ribes nigrum Fruit Extract, Lavandula angustifolia Flower Water, Sodium Hyaluronate, Chamomilla Recutita extract, Calendula officinalis Flower Extract, Lactic Acid, PEG-40 Hydrogenated Castor Oil, Tocopheryl Acetate, Polysorbate 20, Linseed acid, Retinyl Palmitate, Hydroxyethylcellulose.

How to use:

We recommend applying the gel once a day. Before applying, wash your hands and wash the affected area after which:

- Break the closure seal at the end of the cannula of the single-dose applicator
- Insert the single-dose applicator in the vaginal canal
- Select the single-dose applicator until the tube is completely empty
- Discard the single-dose applicator

Do not store the product residue for later use. In the most serious cases lubricate the external part before insertion.

Warnings:

Keep out of reach of children. Single-dose product to be used in a single administration. Store the product properly closed and away from light and heat sources. Do not use the product after the expiry date shown on the package. The expiry date is valid for the correctly stored product. Do not use the product if the package is damaged or opened. The device is not sterile. It is not a contraceptive. Do not use in case of organ lesions. Do not swallow. Avoid contact with eyes. For pregnant and lactating women and girls, the use of the gel should be limited to cases of real need and under the direct supervision of the doctor. Do not use in case of individual hypersensitivity to the components present. Do not use the product in association with quaternary ammonium compounds. The incorrect observance of the warnings, of the methods of use, as well as the re-use of the same single-dose applicator, could compromise the efficacy of the product or cause the aggravation of existing problems or cause the onset of problems that were not previously present. Do not use the product for a period exceeding 30 consecutive days. In case of intact hymen, consult your gynecologist before use. It is important to inform your doctor or pharmacist of any side effects not described in this leaflet.

Side effects:

In normal and reasonably foreseeable conditions of use, undesirable effects specifically related to the use of the product are not known at the moment. Its compatibility has been verified, but in case of irritating or allergic phenomena, suspend its use and contact your doctor / gynecologist.

Packaging:

6 single dose applicators of 5 ml each.

Gruppo Farma Impresa Srl
 Head Office: Via Cipro, 1
 25124 Brescia (BS) - Italy
 Operative Site: Via Il Diestro Corte, snc
 Il Traversa - 81030 Teverola (CE) - Italy