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Borox spray is a medical device for otological use, helpful in specific hygiene of ear cavity as well as in removal of ear wax excess, reducing, thus, the risk of bacterial and fungal proliferation in ear canal, often cause of irritation and inflammation.

Composition:
Aqua, glycerin, propylene glycol, lactic acid, boric acid, sodium docusate, sodium chloride, potassium sorbate, EDTA, sodium benzoate, chamomilla recutita flower water, polysorbate 20, sorbitan oleate.

Dosage:
2-3 sprays per ear cavity, as required or advised by your doctor. It can be used several times daily.

Instructions for use:

- Remove the protective cover of the spray dispenser;
- Rinse the dispenser before use;
- Tilt your torso and head rotation of 45°, introduce the mouth of the dispenser at the entrance of the ear cavity and press one or more times to bring out the desired amount;
- Pull the outer ear upwards and backwards to straighten the external ear canal (CUE);
- Let the solution act a few second, raise the head and blot excess liquid;
- Repeat as needed;
- Clean the dispenser;
- Replace the protective cap dispenser.

Warnings:
Keep away from the reach of children. Do not swallow the product. Do not use in case of tympan perforation. Avoid any contact with eyes. Do not use on damaged skin. Do not use the product if the packaging is damaged. Do not use after the expiry date, the expiry date refers to the product intact and well preserved. In some cases, nebulization may cause a slight transient sense of local irritation and/or dizziness.

Storage:
Store the product properly closed in a dry place, away from light and heat sources.

Side effects:
Possible hypersensitivity to any components. The use of the product, especially if prolonged, may induce sensitivity phenomena. In this case stop the treatment and consult your doctor for a proper therapy. It is important to inform your doctor or pharmacist of any other undesired effects not reported in this leaflet. The use by more people of the same bottle, may lead to the ineffectiveness of the product, the worsening of the problems already in place and the emergence of problems previously not present.

Product Presentation:
Solution in a 50 ml glass bottle provided with spray dispenser.

Manufacturer:
Gruppo Farmaimpresa
Via Cipro, 1
25124 - Brescia (Italy).

FI/CE/18.2/E06
Rev. 21.04.2016

The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.