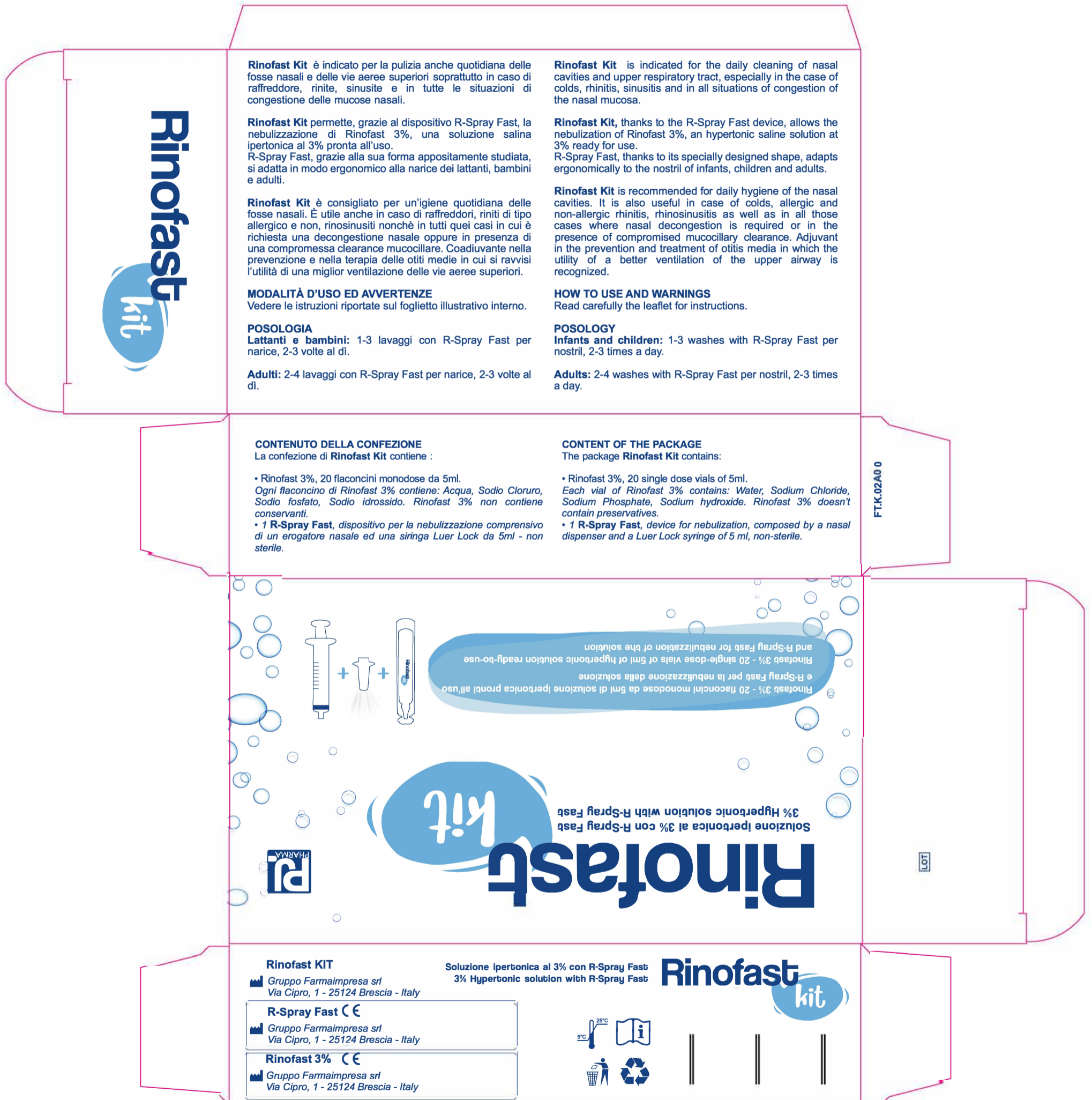
Product: **RINOFAST KIT**  
 Internal code: **FT.K.02**  
 Country:  
 Company: **PJ PHARMA**  
 Operator:

Folded size:  
 Paper:  
 Colors:  
 Finishing:

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**Rinofast Kit**, thanks to the R-Spray Fast device, allows the nebulization of Rinofast 3%, an hypertonic saline solution at 3% ready for use.  
 R-Spray Fast, thanks to its specially designed shape, adapts ergonomically to the nostril of infants, children and adults.

**Rinofast Kit** is recommended for daily hygiene of the nasal cavities. It is also useful in case of colds, allergic and non-allergic rhinitis, rhinosinusitis as well as in all those cases where nasal decongestion is required or in the presence of compromised mucociliary clearance. Adjuvant in the prevention and treatment of otitis media in which the utility of a better ventilation of the upper airway is recognized.

**HOW TO USE AND WARNINGS**  
 Read carefully the leaflet for instructions.

**POSODOGY**  
**Infants and children:** 1-3 washes with R-Spray Fast per nostril, 2-3 times a day.

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**CONTENT OF THE PACKAGE**  
 The package **Rinofast Kit** contains:

- Rinofast 3%, 20 single dose vials of 5ml.

Each vial of Rinofast 3% contains: Water, Sodium Chloride, Sodium Phosphate, Sodium hydroxide. Rinofast 3% doesn't contain preservatives.

- 1 **R-Spray Fast**, device for nebulization, composed by a nasal dispenser and a Luer Lock syringe of 5 ml, non-sterile.

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♻️

LOT

Product: RINOFAST KIT  
 Internal code: FT.K.02  
 Country: -  
 Company: PJPHARMA  
 Operator:

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FT.K.02.F00 rev. del 27/09/2019

# Rinofast kit

3% Hypertonic solution with R-Spray Fast

**Rinofast Kit** is indicated for the daily cleaning of nasal cavities and upper respiratory tract, especially in the case of colds, rhinitis, sinusitis and in all situations of congestion of the nasal mucosa.

**Rinofast Kit**, thanks to the R-Spray Fast device, allows the nebulization of Rinofast 3%, hypertonic saline solution at 3% ready for use. R-Spray Fast thanks to its specially designed shape, adapts ergonomically to the nostril of infants, children and adults.

**Rinofast Kit** is recommended for daily hygiene of the nasal cavities. It is also useful in case of colds, allergic and non-allergic rhinitis, rhinosinusitis as well as in all those cases where nasal decongestion is required or in the presence of compromised mucociliary clearance. It is adjuvant in the prevention and treatment of otitis media in which the utility of a better ventilation of the upper airway is recognized.

#### POSODOLOGY

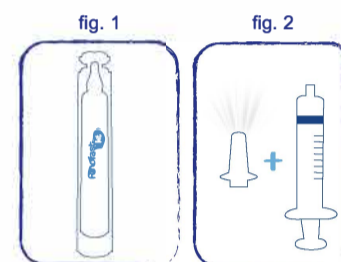
**Infants and children:** 1-3 washes with R-Spray Fast per nostril, 2-3 times a day.

**Adults:** 2-4 washes with R-Spray Fast per nostril, 2-3 times a day.

#### CONTENT OF THE PACKAGE

The package of Rinofast Kit contains:

- Rinofast 3%, 20 single dose vials of 5ml. (fig.1) Each vial of Rinofast 3% contains: Water, Sodium Chloride, Sodium Phosphate, Sodium hydroxide. Rinofast 3% doesn't contain preservatives.
- **1 R-Spray Fast**, device for nebulization, composed by nasal dispenser and a Luer Lock syringe of 5 ml, non-sterile. (fig.2)

#### HOW TO USE

- Remove a single-dose vial and open it by folding the tip along the pre-cut line.
- Insert the tip of the syringe in the mouth of the vial, then draw the solution to be sprayed into the syringe. (fig.3)

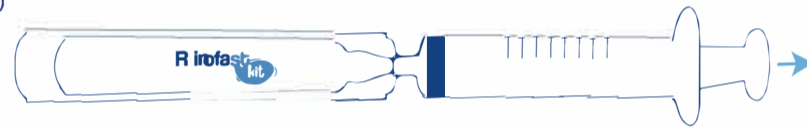
fig. 3

- Assemble R-Spray Fast attaching the syringe, filled with Rinofast 3% to be sprayed, to the dispenser. (fig. 4)

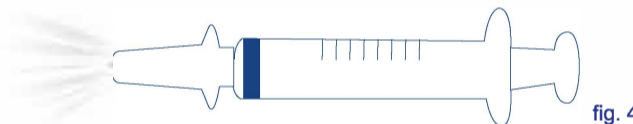
fig. 4

- Remove the cap from the dispenser of R-Spray Fast.
- In infants and children (under 3 years of age), it is necessary to perform the washes lying on your side, keep the head tilted on the other side and gently insert the dispenser of R-Spray Fast into the upper nasal nostril. Energetically press the syringe plunger to spray the solution.
- In infants and adults insert R-Spray Fast in one nasal nostril, staying fully upright and without tilting the head. Draw the solution and simultaneously press the syringe plunger to spray the solution. Leave the syringe plunger before to breathe out.
- Repeat the latter operation also for the other nasal nostril, taking care to equally divide the solution for both nasal nostrils.
- Before replacing the kit into the package, rinse R-Spray Fast.

#### WARNINGS AND PRECAUTIONS FOR USE

- The solution is not injectable.
- Use the product exclusively for the use for which it is intended.
- In children under one years of age and in case of concomitant pharmacological therapy, for the use of the product consult a doctor.
- In case of adverse reactions, stop the treatment and consult a doctor.
- Do not use in case of known hypersensitivity to one or more components of the product.
- If the solution is used for aerosol, it is recommended to follow instructions of the aerosol device.
- Store the product away from light and heat sources.
- Do not use it if the package is damaged.
- Do not use it after the expiry date.
- The expiry date refers to the product properly packaged and correctly stored.
- Before use, check that the R-Spray Fast device is not damaged; if it is damaged, do not use it to spray the solution.
- R-Spray Fast is intended for 1 patient, but it can be used for more nebulizations. For the re-use, clean R-Spray Fast.
- R-Spray Fast can be used for no more than 100 sprays and anyway no more than 6 months.
- The use of R-Spray Fast by children and non-self-sufficient people must take place under the supervision of an adult who has read these instructions.
- Keep out of reach and sight of children.

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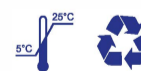
The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.

FT.K.02.F00 rev. on 27/09/2019