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# Lattawin<sup>ovules</sup>

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One ovule contains: Boric Acid, Polycarbophil, Lactic acid, Tocopheryl acetate, Vitamin A Palmitate, 18-beta-glycyrrhetic acid, Tea Tree Oil, Phosphatidylcoline, Semisynthetic Triglycerides.

## DESCRIPTION:

Lattawin Ovules is a medical device that helps recovering the natural vaginal defenses. Containing Boric Acid, Lactic acid, Polycarbophil it helps delete the symptoms due to unbalanced vaginal pH and of endogenous bacterial flora.

Lactic acid, normally present in the vagina, favors an immediate pH correction and facilitates the natural recovery of the delicate vulvovaginal ecosystem.

Boric Acid, thanks to its persistence in the vaginal ecosystem, promotes a reduction of vaginal pH in the long time.

Polycarbophil, once bound to the vaginal mucus, is able to release constantly H<sup>+</sup> ions which help to lower the vaginal environment pH.

## INDICATIONS:

Lattawin Ovules is useful in the prevention and treatment of vaginal affections, also in mycotic character. It helps reducing irritation, burning and itching.

## DOSAGE:

Insert 1 ovule/day in the vagina, preferably in the evening before sleeping, for 5-7 days after the menstrual cycle or according medical prescription.

## METHOD OF USE:

1. Wash vaginal area properly (preferably with a both a vaginal wash external and internal);
2. Open the package and after having washed properly your hands, take one ovule from the blister;
3. Gently insert the ovule deep into the vagina, preferably in supine position, avoiding accidental exit of the ovule out of the vagina.

## STORAGE CONDITIONS:

Store the package properly closed in a dry place, between 5°C and 25°C, away from light and heat sources.

## SIDE EFFECTS:

Known hypersensitivity to any of the ingredients.

## WARNINGS:

Keep out of reach of children. Do not swallow. Do not use if package is damaged. Do not use the product after the expiry date. The expiry date refers to unopened and properly stored product. Do not use the product in case of vaginal lesions and wounds. Do not use the product during menstruation. Do not exceed the suggested dosage without medical advice. The use, especially if prolonged, can lead to sensitiveness events. If this is the case, stop the treatment and consult the doctor in order to start a proper therapy. It is important to inform the doctor or the pharmacist of any undesired effects non reported in the insert leaflet.

## PRECAUTIONS ON USE:

During pregnancy, lactation and for children the use of ovules must be limited only if strictly required and under direct control of your doctor. In case of the intact hymen, before using consult your gynecologist to the specific conditions of use. Monouse device. In case of accidental exit of the ovule out of the vagina, do not re-use the ovule.

## PRESENTATION:

Folding box with 2 blister containing 10 vaginal ovules of 2 g/ovule.

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The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.