Product:  
Internal code:  
Country:  
Company:  
Operator:

Folded size:  
Paper:  
Colors:  
Finishing:

Hypertonic saline solution at 2.5% NaCl and Methyl Sulfonyl Methane (MSM) indicated for daily nasal hygiene. Against the irritation caused by external agents resulting from environmental pollution. Helps to keep free the nose, reduces the dryness moisturizing surface gently. The product can be used in adults and children.

**Dosage:**  
2-3 sprays per nostril, 3-4 times a day or more often as needed.

**Attention:**  
Keep away from the reach of the children. For warnings, storage and instructions for use read carefully the leaflet.

Astuccio / Box:	PAP21	Carta / Paper
Foglietto / Leaflet:	PAP22	Carta / Paper
Etichetta / Label:	PP5	Plastica / Plastic
Flacone / Bottle:	C/IGL95	Vetro / Glass
Erogatore / Dispenser:	OTHER07	Plastica / Plastic
Tappo / Cap:	HDPE2	Plastica / Plastic

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FT/CE/25/51/02 rev. del 16/06/2015

# Rinofast Nasal spray 50ml

HYPERTONIC SALINE SOLUTION AT 2.5% SODIUM CHLORIDE WITH MSM.

Hypertonic saline solution at controlled pH for daily nasal hygiene. Against the irritation caused by external agents resulting from environmental pollution. Thanks to its fluidizing and protective action helps to keep free the nose in case of cooling and also allergic rhinitis. The product should be used in infant under 3 age years old.

**PRODUCT BASED ON:**

Aqua, sodium chloride, Methyl Sulfonyl Methane (MSM), citric acid, potassium citrate and sodium dehydroacetate.

**DOSAGE:**

2-3 spray in each nostril, 3-4 times a day, or more often as needed.

**INSTRUCTIONS FOR USE:**

1. Remove the protective cap from the dispenser.
2. Hold the bottle and the head vertically, gently place the nozzle of the dispenser at the entrance of the nostril.
3. Press with two fingers on the basis of the dispenser.
4. Repeat with the other nostril.
5. Clean the dispenser.
6. Replace the protective cap dispenser.

**WARNINGS:**

Keep away from the reach of the children. In some cases the nebulization can cause a mild and transient sense of local irritation. Avoid spraying in eyes. The use of the same bottle, by multiple subjects, can facilitate the transmission of infections. Do not use in case of epistaxis in progress or just completed.

**STORAGE:**

Store the product at room temperature away from light and heat sources. Do not use the product after the expiration date or if the packaging appear sun damaged. The expiry date refers to the product properly packaged and correctly stored.

**SIDE EFFECTS:**

Possible hypersensitivity to any of the components. The use of the product, especially if prolonged, may induce sensitivity phenomena. In this case stop the treatment and consult your doctor for a proper therapy. It is important to inform your doctor or pharmacist of any other undesired effects not reported in this insert leaflet.

**PRESENTATION:**

Solution in 50 ml glass bottle with spray dispenser.

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The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.

FT/CE/25/51/02 last rev. 16/06/2015