
Product: **TALAVEA OVULES**
 Internal code: **FT.CE.141**
 Country: --
 Company:
 Operator:

Folded size:
 Paper:
 Colors:
 Finishing: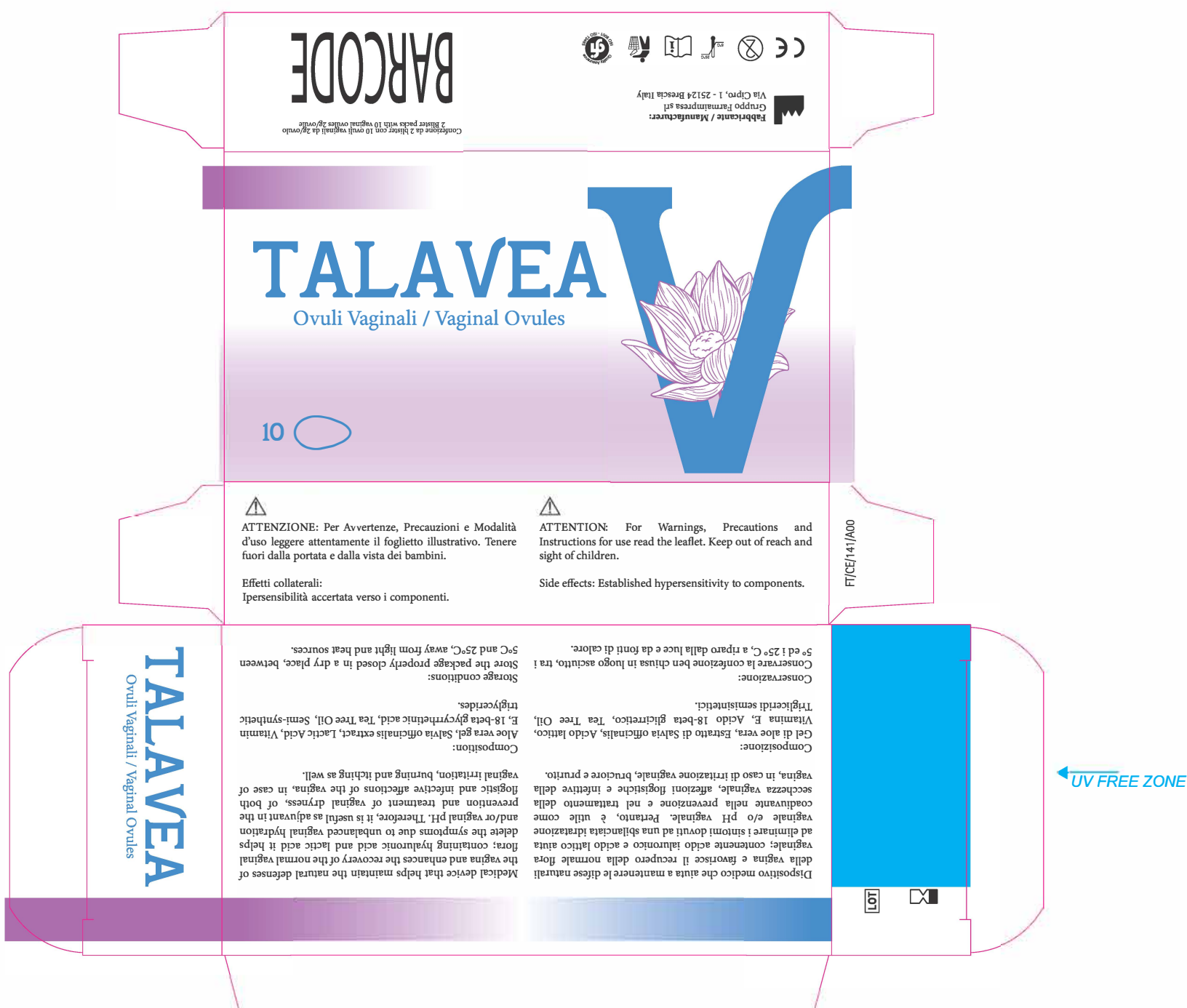

UV FREE ZONE

Product: TALAVEA OVULES
Internal code: FT.CE.141
Country: --
Company:
Operator:

Size flat/folded:
Paper:
Colors:

TALAVEA

Vaginal Ovules

Indications:

Talavea vaginal ovules is a medical device that helps maintain the natural defenses of the vagina and enhances the recovery of the normal vaginal flora; containing hyaluronic acid and lactic acid it helps delete the symptoms due to unbalanced vaginal hydration and/or vaginal pH. Therefore, it is useful as adjuvant in the prevention and treatment of vaginal dryness, of both flogistic and infective affections of the vagina, in case of vaginal irritation, burning and itching as well.

Dosage:

Insert 1 ovule/day in the vagina – preferably in the evening before sleeping - for 5-7 days after the end of menstrual cycle or according medical prescriptions.

Method of Use:

1. Wash vaginal area properly (preferably with both a vaginal wash external and internal);
2. Open the package and after having washed properly your hands, take one ovule from the blister;
3. Gently insert the ovule deep into the vagina, preferably in supine position, avoiding accidental exit of the ovule out of the vagina;

Composition:

Aloe vera gel, Salvia officinalis extract, Lactic Acid, Vitamin E, 18-beta glycyrrhetic acid, Tea Tree Oil, Semi-synthetic triglycerides.

Storage Conditions:

Store the package properly closed in a dry place, between 5°C and 25°C, away from light and heat sources.

Side effects:

Known hypersensitivity to any of the ingredients.

Warnings:

Keep out of reach and sight of children. Do not swallow. Do not use if package has been previously opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. Do not use the product in case of vaginal lesions and wounds. Do not use the product during menstruation. Do not exceed the suggested dosage without medical

advice. The use - especially if prolonge - can lead to sensitiveness events. If this is the case, stop the treatment and consult the doctor in order to start a proper therapy. It is important to inform the doctor or the pharmacist of any undesired effects not reported in the insert leaflet. Do not use the product for more than 30 consecutive days.

Precautions on use:

During pregnancy, lactation and for children the use of ovules must be limited only if strictly required and under direct control of your doctor. In case of the intact hymen, before using consult your gynecologist to the specific conditions of use. Monouse device. In case of accidental exit of the ovule out of the vagina, do not re-use the ovule.

Presentation:

Folding box with two blisters containing 10 vaginal ovules of 2g/ovule.

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