# ILAVIS garze

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# ILAVIS gauzes

PRESERVATIVE FREE - 14 single-use gauzes for adults and children's periocular hygiene

## INDICATIONS:

ILAVIS GARZE is a product for adults and children that is particularly suitable for periocular cleansing, including in pre and post ophthalmic surgery situations. The cleansing is also particularly valuable and effective in association with prescribed specialist therapy in the course of ocular pathological phenomena of allergic, inflammatory and even infectious type (like blepharitis), and for cleansing the eye and surrounding areas.

## PROPERTIES:

Hyaluronic acid sodium salt is one of the fundamental substance making up the extracellular matrix of most of tissues. Thanks to its water retention capacity and its filmogen power, it exerts an hydrating and protective activity.

The functional substances thoroughly cleanse the eye and periocular area making ILAVIS GARZE an excellent hygienic product that can be used for cleansing even during eye therapies prescribed by one's doctor. The special cleansing, moisturizing, protective and emollient activity is guaranteed by the composition of ILAVIS GARZE, which is an extremely gentle product, especially for the most sensitive skin and areas.

## USE AND ADMINISTRATIONS:

Wash your hands properly. Open the sachet and extract the gauze. Cleanse carefully the periocular area with the gauze in order to remove some ocular secretions or any other residuals (smog, dust, powder, etc.) through the soothing emollient function of the product's components. In case of any secretions or particularly persistent occlusion of lachrymal canal, it is suggested to heat the sealed sachet in warm water (40°C) for few minutes and then use the gauze as a warm compress with a gentle massage. Additionally, you can squeeze the content of the gauze on the eye, lean slightly your head, and successively massaging the treated area with the gauze itself upon complete removal of the residuals.

In case of inflammation, it is recommended to cool the sealed sachet in the refrigerator for few hours and then apply the gauze on the eye, keeping it in place for few minutes as a cold compress.

The particular compatibility of the product with the eye area and its composition enables the use of the product even by wearers of contact lenses.

## INGREDIENTS:

100% cotton gauze embided with 3 ml of the following solution: aqua, PEG/PPG — 20/15 dimethicone, chamomilla recutita flower extract, propylene glycol, euphrasia officinalis extract, sodium hyaluronate, ammonium glycyrrhizate, caprylyl/capryl glucoside, sodium cocoyl wheat amino acids, ethylhexylglycerin, phenethyl alcohol, sodium chloride, sodium bicarbonate.

## WARNINGS:

Single use.

Keep out of the reach of children.

Store in a cool dry place.

External use. Do not apply to injured skin or mucosa.

Suspend the treatment in case of clear irritating events or hypersensitivity to the product. The action and the safety of the product are guaranteed exclusively in case of undamaged package.

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