BARCODE

UDI

# MIVAGYN

OVULI VAGINALI • VAGINAL OVULES

Check your country waste disposal 10

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**Mivagn ovules.** Medical device based on Hyaluronic acid sodic salt, 0.5% Polycarbophil 3%, Lactic Acid, 0.2% Phosphatidylcholine, Vitamin E, Tea Tree Oil, Semi-synthetic Triglycerides. **Indications:** Mivagn ovules is a medical device that helps maintain the natural defenses of the vagina and enhances the recovery of the normal vaginal flora; hyaluronic acid and lactic acid helps eliminate the symptoms linked to unbalanced vaginal hydration

and/or vaginal pH, and facilitates the natural recovery of the delicate vulvo-vaginal ecosystem. Polycarbophil interacting with vaginal mucus can constantly release H<sup>+</sup> ions contributing to lower the pH of vaginal environment. Therefore, it is useful as adjuvant in the prevention and treatment of vaginal dryness and also caused by radiotherapy, of both flogistic and infective a ections of the vagina, in case of vaginal irritation, burning and itching as well. The product is compatible with condoms. **Storage Conditions:** Store the package properly closed in a dry place, not exceeding 25°C, away from light and heat sources. **Warnings:** Attention for warnings, precautions and instructions for use carefully read the package leaflet.

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# MIVAGYN

## VAGINAL OVULES

**Mivagyn ovules.** Medical device based on Hyaluronic acid sodic salt, 0,5% Polycarbophil 3%, Lactic Acid, 0,2% Phosphatidylcholine, Vitamin E, Tea Tree Oil, Semi-synthetic Triglycerides. **Description:** Mivagyn ovules is a medical device that helps maintain the natural defenses of the vagina and facilitates the restoration of vaginal flora; Hyaluronic Acid and Lactic Acid help eliminate the symptoms linked to unbalanced vaginal hydration and/or vaginal pH. Hyaluronic Acid in its sodium salt form at a 10mg/ovule dosage exerts an hydrating and lubricating action against vaginal dryness contributes to restore a natural and comfortable vaginal environment. Lactic Acid naturally found in the vagina promotes an immediate pH adjustment and facilitates the natural recovery of the delicate vulvo-vaginal ecosystem. Polycarbophil helps lower the vaginal environment pH by releasing H+ ions while interacting with the vaginal mucus environment. **Indications:** Mivagyn ovules are indicated for the prevention and treatment of vaginal dryness of non-specific character and also caused by radiotherapy, and in case of atrophic vaginitis. Coadjuvant in case of vaginal irritation, burning, itching, hypersensitivity and vaginal disorder of infectious and inflammatory nature. Indicated in women of childbearing age, menopause as well as in perimenopause. The product is compatible with condoms. **Dosage:** Insert 1 ovule/day into the vagina – preferably in the evening before sleeping - for 5-7 days after the end of menstrual cycle or according to medical advice. **Instruction for Use:** 1) Wash the vaginal area properly before application (preferably with both a external and internal vaginal wash); 2) Wash hands carefully before opening the blister an extract one single ovule; 3) Gently insert the ovule deeply into the vagina, preferably in a supine position, avoiding accidental exit of the ovule out of the vagina. **Storage Conditions:** Store the package properly closed in a dry place, not exceeding 25°C, away from light and heat sources. **Side e ect:** Known hypersensitivity to any of the ingredients. **Warnings:** Keep out of reach and sight of children. The product is exclusively for vaginal use. Do not swallow. Avoid contact with eyes. Do not use if package has been previously opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. Do not use the product in case of vaginal lesions and wounds. Do not use the product during menstruation. Do not exceed the suggested dosage without medical advice. The use - especially if prolonged - can lead to sensitiveness events. If this is the case, stop the treatment and consult the doctor in order to start a proper therapy. Do not use in case of individual hypersensitivity to one of the components of the product. It is important to inform the physician or the pharmacist of any undesired e ets non reported in the insert leaflet. Do not use the product for more than 30 consecutive days. In case of serious incident caused by or linked to the medical device, report the incident to the Manufacturer and the Competent Authority. **Precautions on use:** During pregnancy, breastfeeding women and children, since no specific clinical studies have been carried out, the use of ovules must be limited only if strictly required and under direct control of your physician only. In case of the intact hymen, before using consult your gynecologist to the specific conditions of use. Monouse device. In case of accidental exit of the ovule out of the vagina, do not re-use the ovule; reusing the ovule increases the risks of microbiological contaminations and therefore infections. **Drive:** No e ets on the ability to drive vehicles and use machines have been reported. **Interactions and incompatibilities:** The product must not be used in association with quaternary ammonium compounds such as benzalkonium chloride; Interactions with other products are not known, therefore it is advisable to consult your doctor before using product in association with other medical devices or drugs for vaginal use. **Presentation:** Folding box with two blisters containing 5 vaginal ovules each (10 ovules/pack) 2g/ovule.

 **Manufacturer:**  
Gruppo Farmaimpresa srl  
Via Cipro, 1  
25124 Brescia  
www.farmaimpresa.com

0426

The medical device must not be disposed in the wastewaters or household waste. Ask your pharmacist how to dispose the medical devices no longer required. This will help protect the environment.

 FT.MDR.110.18 Rev. del 25/07/2025